




KANGAROO OMNI™ Enteral Feeding Pump

Quick Reference Guide

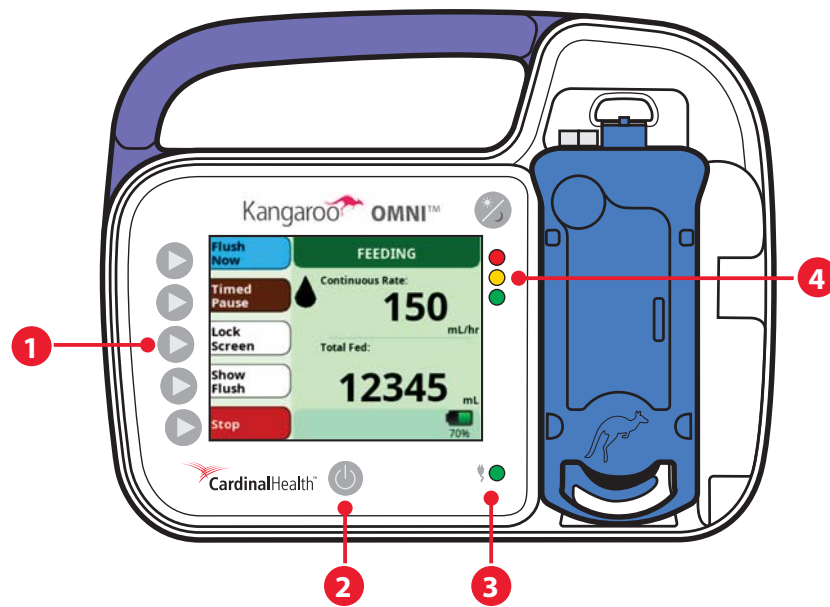

- en-**
- 1. Selection Buttons**
Press a key to select the option that appears next to it on the screen.
 - 2. Power Button**
Press once to power on.
Press and hold to power off.
 - 3. Power Source**
Shines a green light when the pump is connected to AC power.
 - 4. Pump Status**
 - = Warning Alarm
 - = Notice or Caution Alarm
 - = Standby or Feeding

See reverse for more information. ➔

Set Up the Pump

1. Make sure the top of the fluid column is placed at 10 inches (25.4 cm) above the top of the pump.
2. Press **Power** button.
3. Select **Clear Settings** to enter new feed parameters. Select **Keep Settings** to keep old feed parameters.
4. Load Set. Insert from bottom and snap into place.

Prime the Pump

1. Select **Prime**.
2. Select Priming Mode.
3. If you selected Auto Prime: Wait for pump to finish priming.
4. If you selected Manual Prime: Press and hold **Prime Feed** until fluid reaches end of tubing.

Start a New Feed

1. Select **Adjust Rate**.
2. Press the + and - buttons to enter the rate, then select **OK**.
3. When ready to begin feeding, select **Start**.

Pause the Feed

1. To suspend the feeding, select **Timed Pause**.
2. To resume feeding, select **Resume**. Pump will resume on its own after time expires.

Power Down the Pump

1. Press **Power** button.

For additional operations or information, see the Kangaroo OMNI™ Enteral Feeding Pump Instructions for Use.



MR Unsafe – an item that is known to pose hazards in all MR environments. Visit the Kangaroo™ Learning Center at www.mykangaroolearning.com for more information.

Cardinal Health 200, LLC
3651 Birchwood Drive
Waukegan, IL 60085 USA
Rev. A 2023-03 • cardinalhealth.com • HP115204

© 2023 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO, KANGAROO, the Kangaroo LOGO and KANGAROO OMNI are trademarks of Cardinal Health and may be registered in the US and/or in other countries. Patent cardinalhealth.com/patents.